---

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## Bentlon Podo Gold Rotate

65880 (230V) / 65882 (115V)

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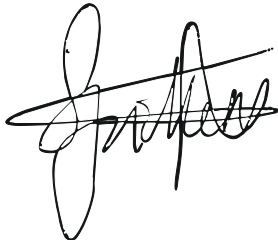
---

## Foreword

We would like to thank you for the trust you have placed in us. We can guarantee that you have made an excellent choice. The design of this product will contribute to the professional image of your institute. You must read this manual carefully as, if your product is used correctly, it will have a longer life and you will not be disappointed in any way. Keep this manual close to hand so that you can consult it quickly if you need to.

**Warning:**

Everything possible has been done to guarantee safe usage, however it is vital that you read this manual carefully right through. If you are in doubt about anything, contact your supplier immediately.

The development department of Dancohr is always on the lookout for the latest technologies and know-how. For this reason there may be slight deviations in the text in this manual or in the operation.

---

# Contents

- 1. GENERAL INFORMATION ..... 4**
  - 1.1 FEATURES BENTLON PODO GOLD ROTATE ..... 4
  - 1.2 BEFORE USING THE CHAIR..... 4
- 2. OPERATING THE PRODUCT ..... 6**
  - 2.1 OPERATING THE CHAIR ..... 6
  - 2.2 ADJUSTMENTS ..... 6
  - 2.3 INCLINATION..... 7
  - 2.4 BACKREST ADJUSTMENT ..... 8
  - 2.5 ROTATION ..... 8
  - 2.6 ARMRESTS..... 9
  - 2.7 HEIGHT ADJUSTMENT ..... 9
  - 2.8 HOME FUNCTION ..... 9
  - 2.9 LEG AND FOOTREST ADJUSTMENTS ..... 10
- 3. MAINTENANCE AND SAFETY MEASURES ..... 11**
  - 3.1 MAINTENANCE ..... 11
  - 3.2 SAFETY MEASURES ..... 11
  - 3.3 DIMENSIONS ..... 12
- 4. TECHNICAL SPECIFICATIONS..... 13**
- 5. RECYCLING INFORMATION ..... 14**
- 6. THE PRODUCT LABEL ..... 15**
- 7. WARRANTY ..... 16**
  - 7.1 WARRANTY POLICY ..... 16
  - 7.2 WARRANTY CLAIMS..... 16
- EC DECLARATION OF CONFORMITY ..... 17**

---

# 1. GENERAL INFORMATION

---

Your product, together with the accessories, is packed in a box that is suitable for normal despatch. However, transport damage may occur. To make sure the product wasn't damaged during transport, we recommend that you check the product as follows:

- (1) Unpack the product and the accessories carefully.
- (2) Check the product and the box for scratches, dents, tears, etc.
- (3) Check all buttons, connections and switches for possible damage.
- (4) Check the accessories for damage.

If the product or accessories have been damaged during transport, you should notify your supplier of this in writing within 15 days of receipt of the product. **Transport damage that is not notified or is notified too late is not included under the guarantee.**

## 1.1 Features Bentlon Podo Gold Rotate

The Bentlon Podo Gold Rotate is a top model amongst the Dancohr pedicure chairs. A foot switch electrically operates the height, the inclination of the seat and the backrest independently from each other. Both leg rests can be adjusted independently in 3 directions. The seat of the chair can be rotated left or right by 0 - 90° and the back rest can be levelled horizontal. Details about the adjustable parts are described read the chapter 'Operating the Product'.

## 1.2 Before using the chair

- After removing the staples that connect the middle section of the box with the bottom, this middle section can easily be taken off.
- The chair now stands free on the case bottom. Now check if there did not occur any damage to the chair during transport. In case of damage immediately call the dealer.
- Now two persons can lift the chair and carry it carefully to its place you had in mind.

In the plastic bag accompanying the chair you find the necessary tools to mount the chair on the floor and to mount the leg rest on the chair.

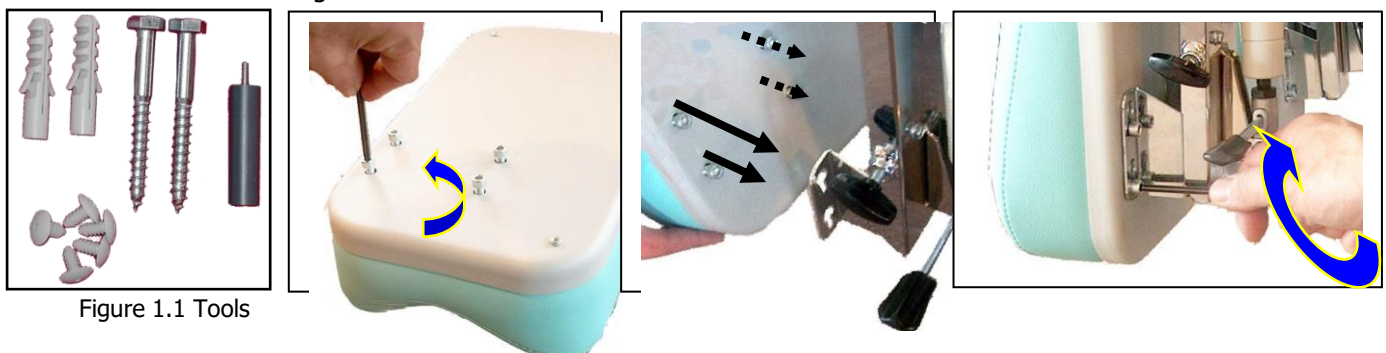
Figure 1.1 Tools

The adjustable feet are can be found at the corners of the chair. (See illustration 1.2). They can be screwed right to make them shorter and left to make them longer. An uneven flooring or otherwise instability of the chair stands can be corrected with these adjustable feet. Adjust the chair to its highest possible position and tilt it sideward to have easy access to the feet.

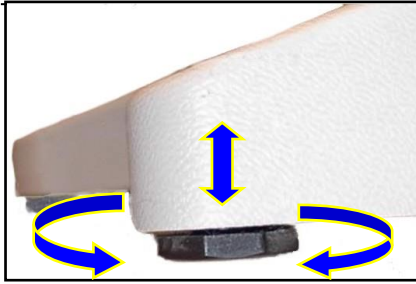
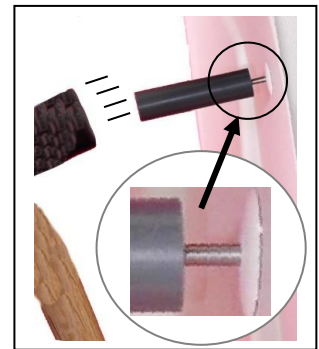
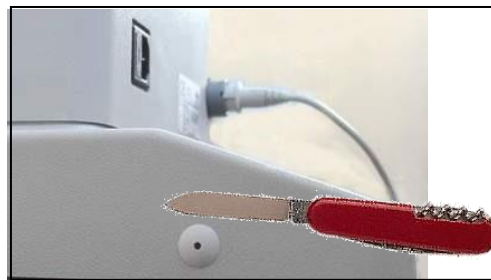
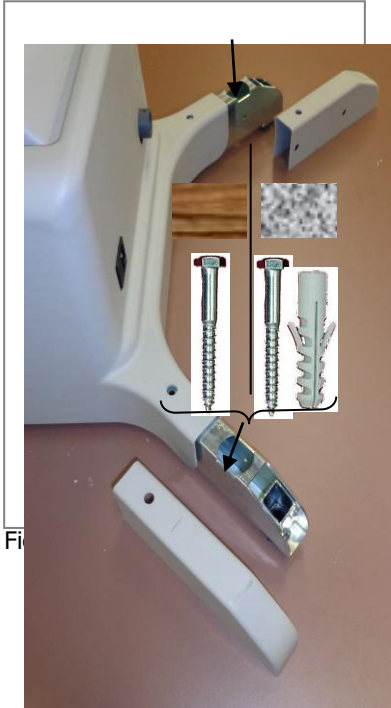
Figure 1.2 Adjustable foot

- To mount the chair, adjust the height to about the level shown in.
- Then carefully cut the plastic plugs of the bottom with a small knife (see figure 1.3)

Fi

be removed.

pieces of the 4 plugs can easily be removed with a tool.

the floor.

on floor only use the two supplied screws.

mounted on a stone or concrete floor first introduce the two supplied wall plugs before mounting the chair with the screws. For drilling into a stone or concrete floor you need a Ø 12 mm stone drill.

After having mounted the chair to the floor, both covers can be refitted to the chair. For this job you need the 4 plugs and the tool

- Stick the supplied tool into one of the plugs.
- With a light stroke of a hammer the plugs are struck into the holes. Repeat this for the other 3 plugs.

The arm supports of the Benton® Podo Platinum are assembled as follows:

Figure 1.4 Fitting the arm supports

Move the metal arm support all the way up to its furthest rear position. Place the upholstery on the underside of the arm. Then press the plastic cover over the upholstery so that it remains entirely in position. To finish, securely tighten the screws provided using a flat screwdriver.

---

## 2. OPERATING THE PRODUCT

---

### 2.1 Operating the chair

On the rear of the chair right is the the socket for the electric foot control. The plug is fitted correctly when it is inserted like shown in fig. 2.1. Make sure the pins are well orientated. The plug needs to be pressed as far as possible into the socket, which may need some effort.

On the left side of the socket for the electric foot control the mains socket is situated. First plug the power cord into the socket and next connect it to a grounded power strip or AC outlet.

Figure 2.1 Inserting plug

**Attention:**

Before connecting the power cord to the mains, check if the mains voltage indicated on the CE label (on the rear the chair) corresponds with the mains voltage in your country or area.

### 2.2 Adjustments

The working of the chair can be adjusted electrically with the switches of the foot control (see figure 2.2). Keep the switches pushed down until the desired position is obtained.

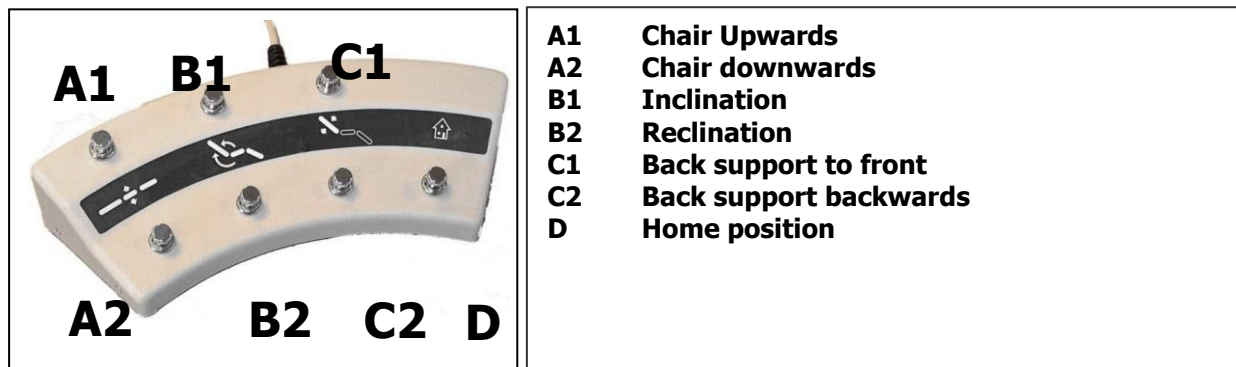
Figure 2.2 Adjustments

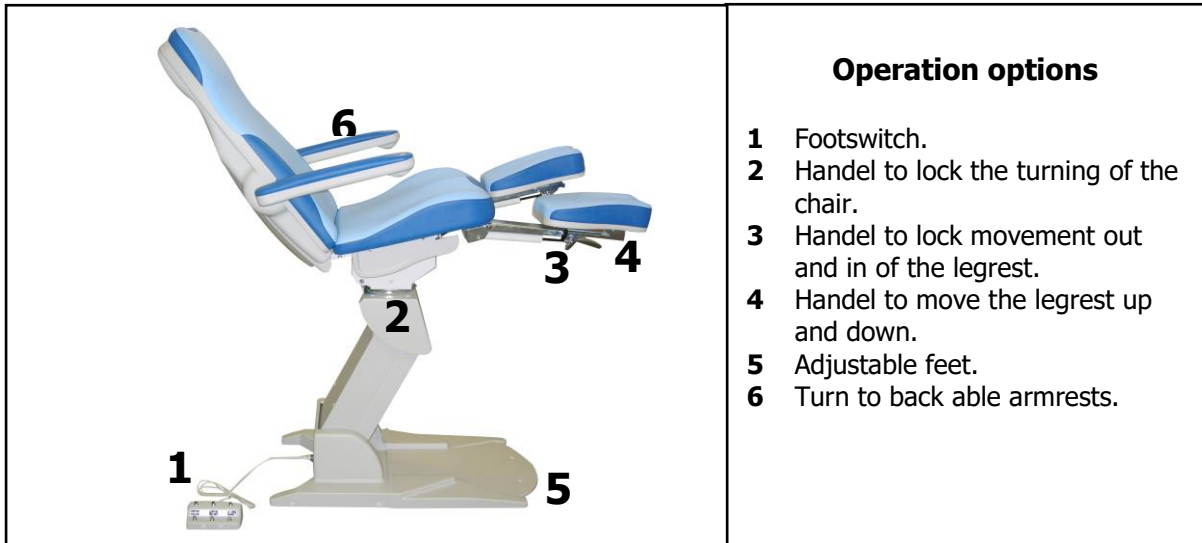
Figure 2.3 Operating options

## 2.3 Inclination

The tilt adjustment can be operated electrically with the two switches in the middle of the foot control. Figure 2.4 shows how the sections move.

Figure 2.4 Moving sections

---

## 2.4 Backrest adjustment

The backrest adjustment of the chair is controlled electrically with the two right switches on the foot control. Figure 2.5 shows how the section moves.

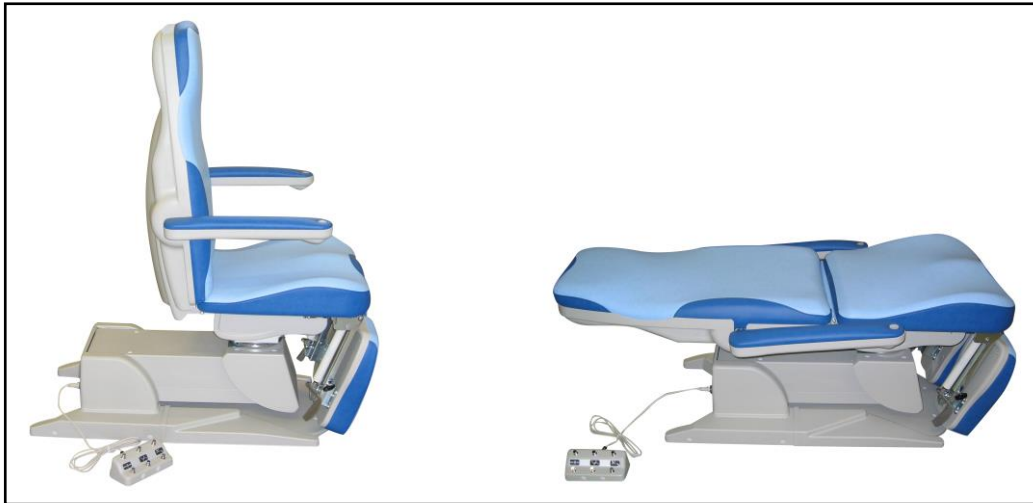
Figure 2.5 Backrest adjustment

## 2.5 Rotation

The seat of the chair can be rotated left or right by 0 - 90°. Underneath the seat the lever is situated. To lock the seat, turn the lever to the right. The chair cannot be rotated in this situation. Unscrew the lever underneath the seat to enable the rotation right or left by 90°. A quarter turn of the lever is sufficient. In case you turned it too wide, making it uneasy to operate, first screw home the lever and then push it outward (see the arrow on fig. 2.6). Keep the lever meanwhile turning it to the desired position. When you relinquish the lever it will stay in the right position. Please make sure the handle is always directed downwards to avoid a collision.

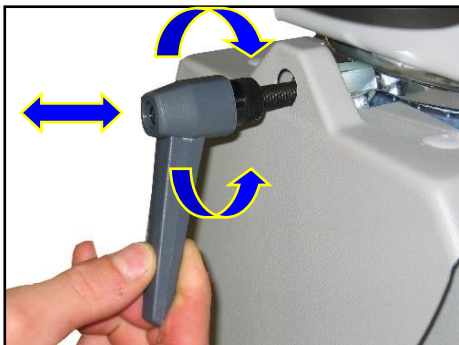
Figure 2.6 Rotation

**Caution:**

Never rotate the chair, until you made sure, that it is mounted to the floor as prescribed.

## 2.6 Armrests

As a rule the armrests remain parallel to the seat. When the chair is in completely horizontal position, the armrests to front are on the same level as the seat. It allows you to use the full width of the chair. With the chair is at its full length it's also possible to pivot the armrests backwards below the level of the backrest. This is recommendable for back massage. For a client lying facedown it means he can rest his arms in a natural position. The little hole at the extreme point of the armrests offers you the possibility to fit the optionally available manicure bowl (fig. 2.7). To allow easy access for your client, the armrests can be pivoted backwards.

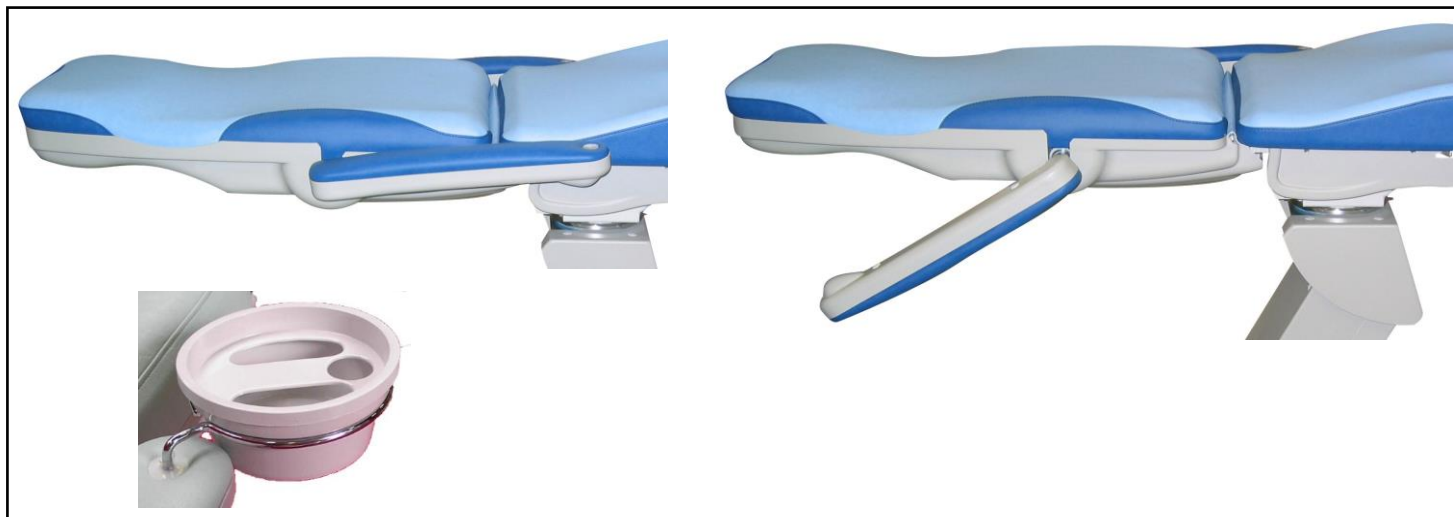
Figure 2.7

## 2.7 Height Adjustment

The working of the height of the chair can be adjusted electrically with the two switches on the left of the foot control (fig. 2.2). Keep the switches pushed down until the desired position is obtained (fig. 2.8).

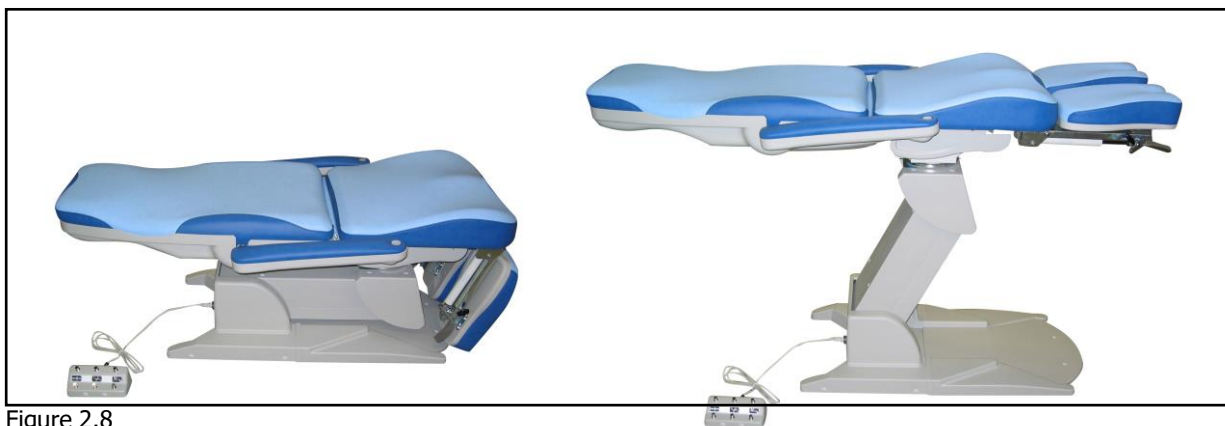
Figure 2.8

## 2.8 Home function

The chair can be adjusted with a single button on the foot pedal to the home position (Figure 2.2). In this position, one can easily get up from the chair and sit in the chair.

## 2.9 Leg and footrest adjustments

The leg rest is three-way adjustable:

**Length:** The leg rest are in a simple manner more than 20 centimeters extendable (fig. 2.9). Loosen the button on the exterior of the leg rest and pull or push the leg into the desired position. Turn the button again easily, in order to tighten the leg rest. It is not necessary to tighten this button firmly.

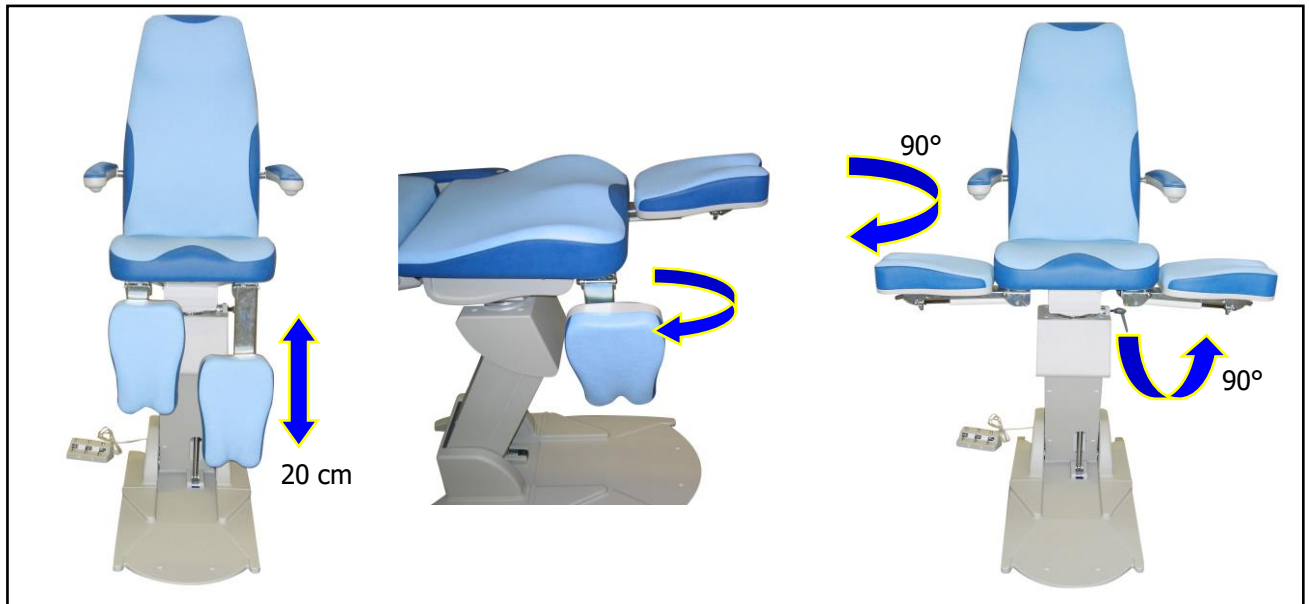
Figure 2.9

**Height:** To adjust the leg rest operate release the lever (fig 2.10) toward the arrow and simply pull the leg rest highly. For lowering you operate the release lever again toward the arrow. By pressure lower the leg rest release the lever as soon as the desired position of the leg rest is reached.

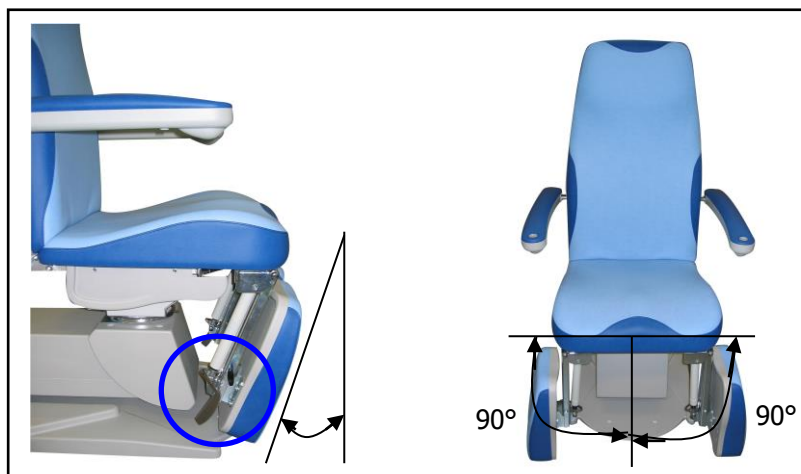
Figure 2.10

**Sideways:** In order to adjust the leg rest up to 90° sideways, just push the leg rest outward or back (figure 2.10). The leg rest turned outward facilitate the accessibility of the inside of the foot. It is easy to enter the chair for older and shorter people.

---

## 3. MAINTENANCE AND SAFETY MEASURES

---

### ***3.1 Maintenance***

- Remove stains immediately as otherwise they may become permanent or damage the vinyl. Be careful in particular with suntan oil, insect repellents and make-up. Test stain removal first in an inconspicuous place.
- Wipe the upholstery and housings down regularly with a damp soft cloth.
- Remove stains with a solution of soft household soap and use a soft cloth (if necessary a soft brush). Remove the remains of the soap with a clean damp cloth and then dry the chair with a towel. Use luke-warm water and do not saturate the upholstery. Make sure that you dry the seams properly.
- Stubborn stains can generally be removed with isopropylalcohol from the chemist's; make sure that you wipe it off with a damp cloth.
- If the stains cannot be removed from the upholstery, contact your supplier. NEVER use abrasives, pan sponges, bleach or solvents such as methylated spirits, benzene, acetone, etc. These may make the upholstery or plastic housings dull or damage them.
- Be careful with water nearby the electric foot control and the electric mains and make sure no water can run into the column.
- Although the chair does not need maintenance, a regular check-up every two years is advisable. Contact your dealer for this.
- When operating the chair is accompanied with a squeezing sound, this can be put right by dropping a bit of thin oil, (sewing machine oil) on that spot. If this problem cannot be solved with this, please contact your dealer.

### ***3.2 Safety measures***

These chairs are very well-made and have been manufactured to last many years. For optimum use it is important that the chairs are used according to these instructions.

- Never sit on the back or leg support. If you do this, it may result in serious damage to the supporting frame of the chair.
- The bottom of the client should always be on the seat of the chair, even when getting into a chair in the massage position. This guarantees the stability of the whole unit.
- Do not stand on the foot operation cable and regularly check the leads for breakage. Replace these if necessary.
- Make sure, that the power cord of the chair is in a correct condition. Check it regularly und have it replaced in case of damage.

### 3.3 Dimensions

For the dimensions (in millimetres) see figure 3.1.

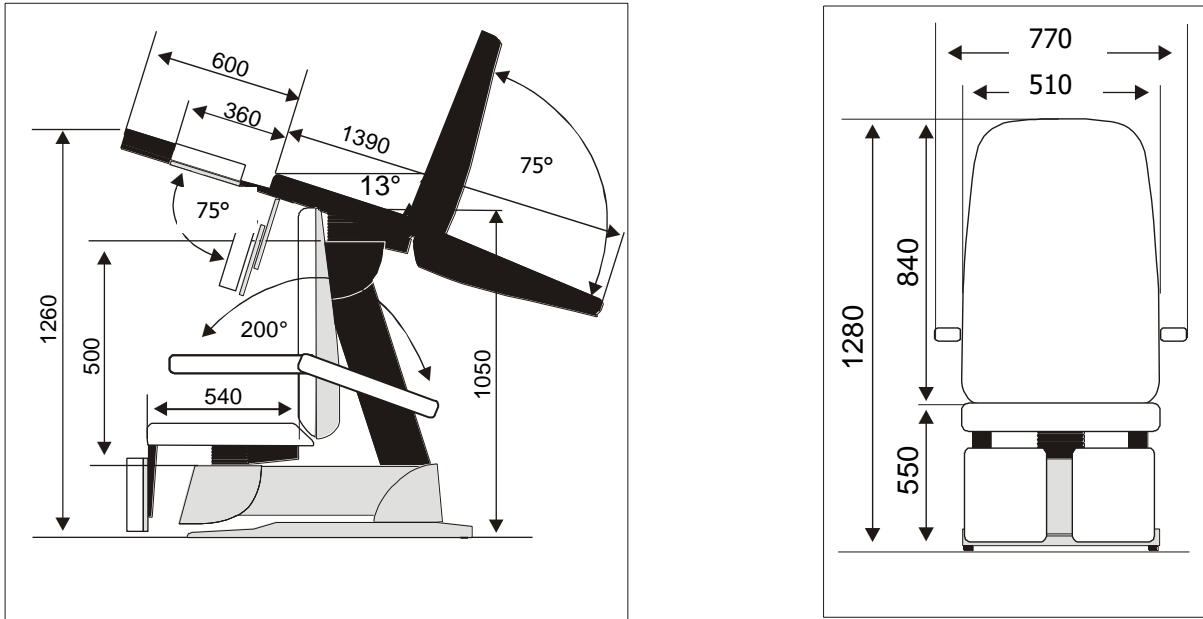
Figure 3.1 Dimensions

---

## 4. TECHNICAL SPECIFICATIONS

---

	<b>230 volts</b>	<b>115 volts</b>
Dimensions (Lx B x H) Max.-Min. (cm)	178-200 x 77 x 55-107	178-200 x 77 x 55-107
Seat width	53 cm	53 cm
Weight	74 kg	74 kg
Connection value	AC 220-240V; 50/60 Hz	AC 110-130V; 50/60 Hz
Maximum <b>even</b> load	200 kg	200 kg
Power consumption	96 Watt	96 Watt

---

## 5. RECYCLING INFORMATION

---

Dancohr has drawn up technical and economically feasible targets to optimise the ecological performance of the products, service and activities.

During the planning, design and production phases Dancohr places emphasis on the manufacture of equipment that can be easily and cheaply recycled. In addition, Dancohr is heavily involved in participation in return programmes and in national cooperation projects to draw up and implement recycling procedures for the recycling of electrical equipment on a larger scale.

The symbol for the waste electrical and electronic equipment (WEEE) directive is given below.

The use of the above symbol for waste electrical and electronic equipment (WEEE) indicates that this product may not be disposed of as household waste. This is laid down in the European Directive 2002/96/EC relating to the disposal of used electrical and electronic equipment. By making sure that this product is disposed of correctly, you are contributing to the protection of the environment. You, as the owner, are held responsible for the disposal of the product to the closest collection point for electrical and electronic waste. For more detailed information on recycling this product, consult your local authority, the household waste service or the shop you purchased this product from.

## 6. THE PRODUCT LABEL

The product label contains the following data:

*Label for 230V product*

 Dancohr Corporation Marconilaan 8 6003 DD Weert The Netherlands www.dancohr.com		
REF		
		
special info		
Fuse external	SN	
Voltage	   	
Frequency		
Power		

*Label for product other than 230V*

 Dancohr Corporation Marconilaan 8 6003 DD Weert The Netherlands www.dancohr.com		
REF		
		
special info		
Fuse external	SN	
Voltage	   	
Frequency		
Power		

Address details of the manufacturer.

Reference number and name of the product.

Date of manufacture (month year / serial number of that day).

Special info: Extra information.  
 Fuse external: Rating of the fuse at the supply connection.  
 Voltage: Required voltage level for proper working in Volts.  
 Frequency: Required frequency for proper working in Hertz.  
 Power: Power consumption in watts.

Serial number of the product.

Warning, read the documents supplied before use.

Fragile, handle with care.

Keep the product dry.

This machine must be disposed of in an environmentally responsible manner.

This label guarantees that the product has been constructed according to the latest EU directives. Do not remove this sticker. The details stated on the sticker are important for the guarantee and you will need to provide these details should you require information about the machine from your supplier.

---

## **7. WARRANTY**

---

### ***7.1 Warranty policy***

This warranty policy is derived from the General Terms and Conditions filed with the Chamber of Commerce.

The Supplier extends a 24-month general warranty on this product with regard to defects and a maximum of 60 months for structural defects. The warranty period begins on the date of invoice. Any attempts by persons or organisations that have not been designated as qualified by the Supplier to repair, change or modify the product beyond the actions and operations described in the user manual shall result in the expiration of the warranty and of any responsibility by the Supplier. The Supplier shall not be responsible in any way for the damages that result from these actions.

The Supplier and its representation cannot be held responsible for any damage, (physical) injury or poor functioning of the product caused by improper, negligent and/or incompetent use as a result of insufficient study of the instructions as described in the user manual or as obtained from the training course.

Discolouration of upholstery and other parts as a result of non-colour-fast clothing, aggressive cleaning products or oil-containing products is not covered by the warranty.

The following conditions must be met in order to ensure the optimal operation of the product:

- The electrical installation of the relevant room must comply with the applicable, statutory regulations;
- Upon purchase, the presence of all accessories, parts and the user manual must be verified. Notification of any missing items or of damage to one or more of the parts delivered must be submitted to the Supplier within 7 days of the date of invoice. In the absence, within the stated time limit, of verification and/or a submission of a notification of damage to or missing items from the necessary devices, the warranty shall expire.
- The product must be used by authorised persons according to the instructions as stated in this user manual or as obtained in the training course.
- The product must be used for the purposes for which it was designed;
- The maintenance of the product must occur regularly and must be carried out in the prescribed manner;
- The statutory regulations with regard to the use of this machine must be observed.

The Service Department of the Supplier provides a binding assessment of the compliance to the conditions related to repair under warranty.

### ***7.2 Warranty Claims***

In the event of defects or structural defects, the customer must apply to the Supplier. When doing so, the following contact details may be used:

Dancohr Corporation BV  
Marconilaan 8  
6003 DD Weert  
The Netherlands  
Tel: +31 (0)495 434 600  
E-mail : [info@dancohr.com](mailto:info@dancohr.com)  
Website : [www.dancohr.com](http://www.dancohr.com)

This text is published by Dancohr Corporation BV. The Supplier retains the right to change the specifications of the device in conformity with the technical developments of the product.

## EC DECLARATION OF CONFORMITY

---

Dancohr Corporation  
8 Marconilaan  
6003 DD Weert  
The Netherlands  
Tel. +31 495 434600

Declares hereby that this product fulfils the conditions according to the European guidelines and this unit:

1. fulfils the conditions stated in the Machine guideline
  - 98/37/EC
  
2. fulfils the conditions stated in the Low-voltage guideline
  - 2006/95/EC
  
3. fulfils the conditions stated in the Electromagnetic compatibility guideline
  - 2004/108/EC
  - 92/31/EEC

Place: Weert, Date: 20-10-2016

A handwritten signature in black ink, appearing to read 'R. Spitters', is written over a horizontal line.

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