

**INSTRUCTIONS FOR USE OF
Strap Arm Sling, 5.5cm*12m**

LS4103

Intended use

The sling is used to immobilize and/or support the wrist, arm and/or shoulder.

Product Features

- Made of quilted strong blue foam material;
- Easy to cut to size;
- Can be fastened with included Y-shaped Velcro strips.


Indications

- Wrist, elbow or shoulder trauma;
- Arm support in combination with a cast.


Contraindications / Side-effects

None known at present.

Fitting instructions

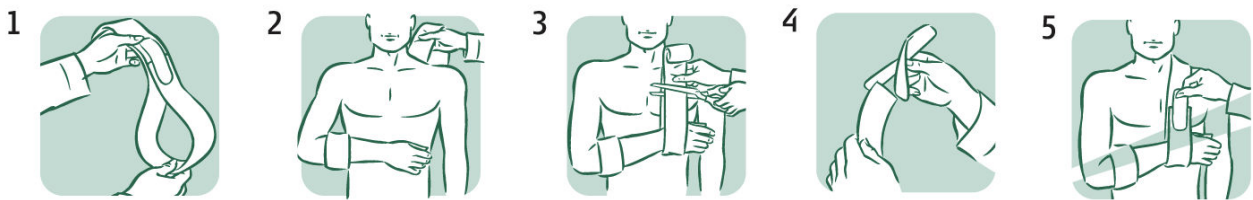

Applications

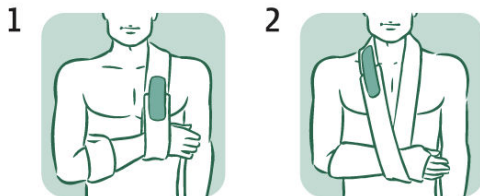

- The brace is recommended for one patient use only.
- The brace is not intended to be applied directly on breached skin.
- Respect the cleaning instructions.
- Remove device before showering.



If allergic reactions occur due to the use of the brace, please consult your health care practitioner.

Any serious incidents that occurred in relation to this device should be reported to the manufacturer and the competent authority of the member state in which the user and/or patient is established.

Used materials

40% Brushed nylon fabric; 40% PU foam; 8% Nylon hook; 2% cotton thread.

Cleaning

Wash by hand in mild, cold soapy water. Rinse thoroughly.



- Do not bleach.
- Do not wring out.
- Do not iron.
- Do not put the device in a washing machine or dryer.

Used Symbols




Medical device



Single patient multiple use



Shanghai Leison Medical & Healthcare Products Co.,Ltd
No.99,West Puhong Road,Pudong Area,Shanghai,201324,China

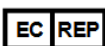

SUNGO Europe B.V.
Olympisch Stadion 24, 1076DE Amsterdam,Netherlands

